



Monitor FallGuard CordLess Moniteur à prévention de chute Smart sans fil



SMART GBT-W1 and GCT-W1 Wireless Bed Pad User's Guide

For safety reasons, it is recommended to replace the pad after one year of use.

This pad is designed to be used with cordless fall management monitors. Neither pads nor wireless monitors require cables to work together. The wireless technology used by the patent pending pad relies on a transmitter embedded in its sleeve to communicate with the monitor.

Warning: The manufacturer does not guarantee that this device will prevent users from falling or attempting to escape.

Please note that this pad is a standard model and does not have an internal clock. It is compatible with monitors of the "Change pad indicator" series, but the "Change pad" indicator will not signal the end of the warranty period for the pad. This system is designed to work only with a pad equipped with an internal clock. For further information, please contact: 1-800-361-3537

Commissioning

1. Insert the batteries into the SMART Wireless Waste Management Monitor. With the transmitter nearby, press the "Program" button on the back of the monitor. It will beep three times to indicate that it is in programming mode and the "In use" light will turn on.
2. Activate the transmitter. The monitor will emit a long tone indicating that programming is complete. Replace the transmitter in the pad sleeve.
3. Apply pressure on the wireless pad to activate the monitor. This will give two "beeps" confirming that the pad and monitor are working properly. The "In use" light will flash as long as the pressure is maintained. When the pressure is released, the monitor will trigger an alarm. Press the "Reset" button to stop the alarm.
4. When the pad needs to be replaced, remove the transmitter from the sleeve. Slide the transmitter slider to "Off" and keep the transmitter until you can use it with a new wireless pad.
5. To ensure the stability of the pad on the bed, it is recommended to use the optional mounting system.

Maintenance

- When using the pad with a foam or gel topper, always check that the system is working properly.
- To limit the risk of infection, it is advisable to use a new pad for each patient.
- Clean the pad with a disinfectant wipe or antibacterial product.
- Do not fold the pad.
- Do not immerse the pad in any liquid or cleaning solution.
- Do not expose the pad to high temperatures or place in a sterilizer.



Monitor FallGuard CordLess Moniteur à prévention de chute Smart sans fil



Troubleshooting

- Check the "Pad signal" light on the monitor. The "Status / In use" light should blink to indicate that the pad is working properly. If the "Check pad" light flashes, it may mean that the pad is out of range (approximately 7 metres) or that the transmitter battery is low.
- Make sure that the transmitter cursor is in the "On" position and the red light flashes every 20 seconds, confirming that it is working properly. If not, please contact us at 1-800-361-3537.
- Check the monitor power supply. If you are using a battery and the "Low battery" light is on, replace the battery. If an AC adapter is used, make sure it is plugged in and the wall outlet is working.

If the problem persists, contact us with the monitor, pad and transmitter available.

WARNING

Failure to comply with safety instructions may result in serious accidents, including the death of the user. This device is not suitable for all users and may require the use of additional equipment. It does not replace visual surveillance. The manufacturer does not guarantee that this device will prevent falls or attempts to escape. Pads and monitors may fail. Ensure that the user cannot become entangled or choked with a cable. This device is designed to assist in the management of waste. Check its proper functioning before each use. Read the instructions and warnings carefully before use.

LEGAL NOTICE

SMART CAREGIVER CORPORATION pads and monitors are designed for user installation. It is therefore its responsibility to ensure the correct assembly and necessary verifications. This device does not replace staff supervision and requires:

- o A patient supervision
- o Adequate training of staff in the management of falls and escapes
- o A check of the device before each use

The seller limits its liability to the refund or replacement of products. It cannot be held responsible for any damage resulting from their use.

LIMITED WARRANTY

The seller warrants that the product is free from defects in workmanship for the period specified on the pad, starting from the date of purchase. No other implied warranties apply.

No statement, verbal or written, by the seller constitutes an additional guarantee. The buyer is responsible for the performance of the product after installation. In the event of a defect, only the costs of installation, maintenance or repair will be borne by the buyer and not by the manufacturer, distributor or retailer.

Manufactured by: Smart Caregiver Corporation 1229N. Mc Dowell Blvd Petaluma, California 949